

Cleaning and Sterilisation Steps for Metal Ring-Grip



1 Add detergent to warm water, then insert the tube attached to the instrument and flush thoroughly.



2 Reflush the instrument with clean water.



3 Remove the tube and insert a small quantity of vegetable oil in the inlet fitting.



4 Fill a deep pot with clean water. Attach tube to instrument and pump handle to draw water in to fill the cylinder.



5 Suspend the whole instrument so it does not touch the base of the pot, add the needles and boil for 10 - 20 minutes. ***Failing to suspend the instrument may result in damage to the instrument.***



6 Remove instrument from the pot, then wrap a clean cloth around the handle and pump dry.

prodigy
instruments

Prodigy Instruments Pty Ltd ABN: 52 609 855 576

Postal Address PO Box 120, Mount Kuring-Gai, NSW 2080 Australia

Email matthew@prodigyinstruments.com.au

Office +61 403 454 354

Web prodigyinstruments.com.au